


# **IOL INJECTOR KIT**

**Bulk Purchase** 





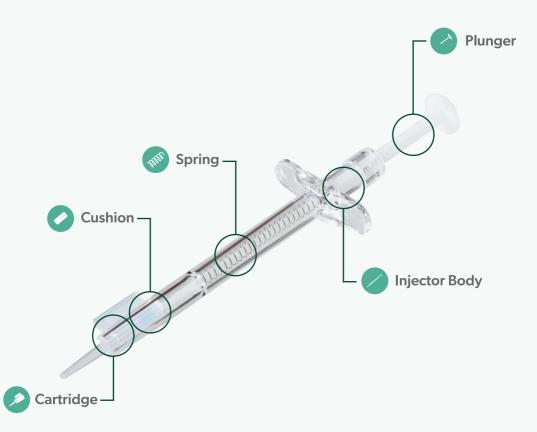





# **COMPONENTS** & DESCRIPTION



## **Spring**

Allows push back for safety & controlled delivery



#### Plunger

Engineered design & robust material for secure alignment



#### **Injector Body**

Designed for both mono & bi-manual injection procedure & allows visibility during delivery process



#### Cushion

Safety stopper for a controlled start



## Cartridge

Cartridge size from 2.2mm to 2.4mm

# **ASSEMBLY INSTRUCTIONS**



Put the **Spring** on the Plunger



Insert the Plunger into the **Injector Body** 



Insert the **Cushion** on top of the **Plunger** 



Slide the Intraocular Lens into the Cartridge



Place the **Cartridge** into the **Injector** and **click** it in place



See step-by-step assembly images

## **HOW TO LOAD**

See the process of loading an IOL into the ECO FLY injector for surgery



Prices available on request and subject to volume







