


ERGO  
TOUCH  
FLY

# IOL INJECTOR KIT

Bulk Purchase

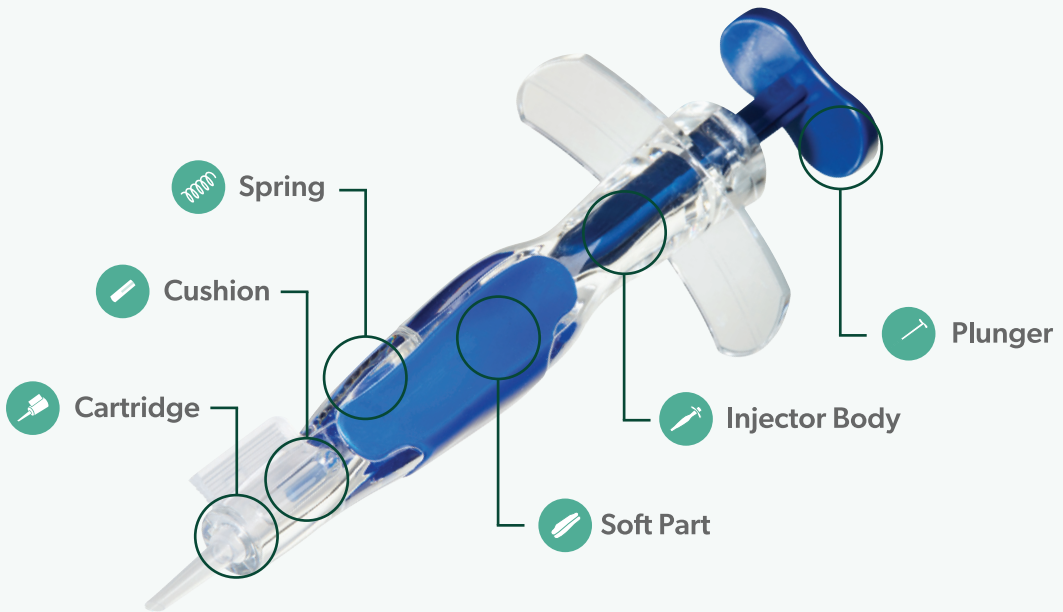

## COMPONENTS & DESCRIPTION



### Spring

Allows push back for safety & controlled delivery



### Plunger

Engineered design & robust material for secure alignment



### Soft Part

Innovative anti-slip texture for safety (when using wet gloves) & comfort of use



### Injector Body

Designed for both mono & bi-manual injection procedure & allows visibility during delivery process



### Cushion

Safety stopper for a controlled start



### Cartridge

Cartridge size from 1.8mm to 2.6mm

## ASSEMBLY INSTRUCTIONS

- 1 Put the **Spring** on the **Plunger**
  - 2 Put the **Soft Part** on the **Injector Body**
  - 3 Insert the **Plunger** into the **Injector Body**
  - 4 Push the **Plunger** all the way into the **Injector Body**
  - 5 Insert the **Cushion** on top of the **Plunger**
  - 6 Place the **Intraocular Lens** into the **Cartridge**
  - 7 Place the **Cartridge** into the **Injector** and **click** it in place
- > See step-by-step assembly images

## HOW TO LOAD

See the process of loading an IOL into the ErgoTouch FLY injector for surgery



Prices available on request and subject to volume

